



vac-clean uv[®]

Rechargeable mini vacuum cleaner for all hearing aids.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 3
DE	GEBRAUCHSANWEISUNG	4 - 5
FR	MODE D'EMPLOI	6 - 7
ES	MANUAL DE INSTRUCCIONES	8 - 9
PT	MANUAL DE INSTRUÇÕES	10 - 11
IT	ISTRUZIONI PER L'USO	12 - 13
NL	GEBRUIKSAANWIJZING	14 - 15
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	16 - 17
TR	KULLANIM TALİMATI	18 - 19
DK	BRUGERVEJLEDNING	20 - 21
SE	BRUKSANVISNING	22 - 23
NO	BRUKSINSTRUKSJONER	24 - 25
PL	INSTRUKCJA OBSŁUGI	26 - 27
RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ	28 - 29
CN	使用说明 (简体中文)	30 - 31
JP	取扱説明書	32 - 33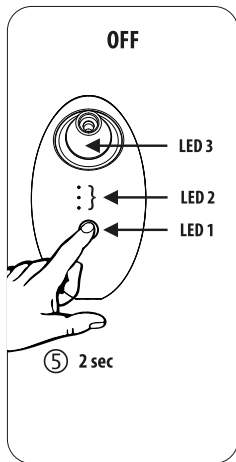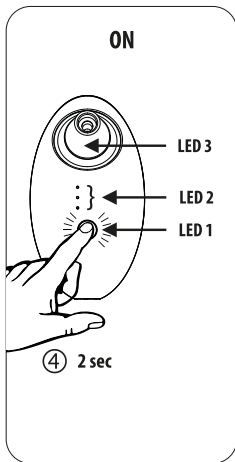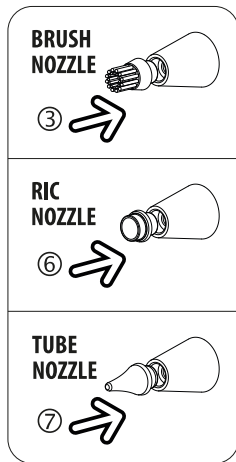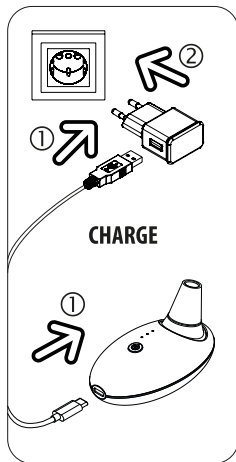

EN INSTRUCTION FOR USE

Initial operation

1. Plug the USB cable into a USB power supply unit (5V/1A) and the USB-C plug into the USB-C socket of your vac-clean uv®.
 2. Then plug the power supply into a mains socket.
 - The vac-clean uv® is charged in approx. 2 hours and ready for operation for approx. 1.5 hours
- ! CAUTION: Operation is not possible during the charging process.

ⓘ LED 1 lights up red

ⓘ LED 1 lights up blue

Operation with the brush attachment nozzle

3. Insert the brush attachment nozzle into the base attachment of the vac-clean uv®.
4. Press button 1 for at least 2 seconds and the vacuum motor starts at the strongest suction level (ooo).
 - The UV light with 450nm starts automatically with an antimicrobial effect
 - Take the previously well-dried hearing aid in your hand and move the brush evenly with light pressure over all sides of the hearing aid to loosen and vacuum up dried cerumen and dirt
5. when cleaning is finished, switch off the device by pressing button 1 for at least 2 seconds.

ⓘ LED 1 and LED 2 (ooo) light up

ⓘ LED 3 lights up

ⓘ LED 1, LED 2 (ooo) and LED 3 turn off

Operation with the RIC attachment nozzle

6. Insert the RIC attachment nozzle into the base attachment of the vac-clean uv® and proceed according to points 3 to 5.
 - Take the RIC earpiece with domes or earpiece of your hearing aid in your hand and directly join the cavity of the RIC attachment nozzle to suck off cerumen and dirt from the openings. If residue remains on the hearing aid, briefly spray cleaning spray and repeat procedure

Operation with the tube attachment nozzle

7. Insert the tube attachment nozzle into the base attachment of the vac-clean uv[®] and proceed according to points 3. to 5.
 - Guide the tip of the tube attachment nozzle to the dirty mini openings of your hearing aids, such as the receiver, microphone, ventilation hole (vent), to suck off cerumen and dirt from the openings. If any residue remains on the hearing aid, briefly spray on cleaning spray and repeat the process

Maintenance and operation

To clean your vac-clean uv[®] we recommend wiping it with damp cleaning cloths. Unscrew the basic attachment with the brush attachment nozzle, RIC attachment nozzle or tube attachment nozzle after use and clean them in an ultrasonic bath (e.g. u-sonic 2). Then dry well under a drying cap (e.g. dry-cap uv 3[®]). The filter in the base attachment should be replaced depending on the degree of contamination. Please also observe the care and operating instructions of your hearing aids. The vac-clean uv[®] is very energy-saving. The lithium-ion battery only needs to be recharged when it is empty (see Initial operation) ⓘ LED 1 lights up red. Your vac-clean uv[®] has 3 suction levels (ooo / oo / o) with different strengths. By briefly pressing the key, you can switch between the suction levels in operation as required.

! Warnings

Avoid contact with water! The vac-clean uv[®] must not be operated near wash basins or similar! Avoid mechanical stress due to impact and falling! The vac-clean uv[®] must not be operated near fire or easily combustible objects! In case of malfunctions, take the device to the dealer! Do not disassemble the unit! Children must be supervised to ensure that they do not use the device as a toy! This device is not intended to be used by persons with reduced physical, sensory or mental capabilities, or lack of experience and/or knowledge, unless they have been given supervision or instruction concerning use of the device by a person responsible for their safety!

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© 2010-2022 flow-med GmbH. All rights reserved.
flow-med GmbH